



# Bone Scraper


For safe and effective collection of autologous bone

The bone scraper allows safe and effective collection of autologous cortical bone fragments for use in regenerative procedures in oral surgery.

Thanks to its round cutting blade, the bone scraper can be applied to surfaces with different shapes (flat, concave, or convex).

## Product features

- The capability to rotate the blade 360-degree allows four sharp edges, which maximizes bone collection
- Ergonomic handle for comfortable grip and control during utilization
- Easy and quick bone harvesting for improved patient comfort

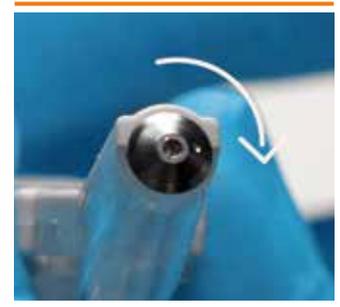

Original size

## Application



Firmly press the front end of the device against the bone surface and pull backwards.



Gently push the transparent chamber forward with your finger and slide it open to collect the harvested bone chips with an instrument.



If the blade becomes dull, use a 1.2 mm hex screwdriver to rotate it 90-degree and generate a new sharp edge.

## Order information

Art. No	Article	Package size
<b>BP2001.1005</b>	Bone Scraper, sterile, single use	5 units

Our services and deliveries are provided exclusively on the basis of the General Terms and Conditions.



### BioHorizons Camlog Benelux B.V.

Pelmolenlaan 1 | 3447 GW Woerden | Nederland  
info@camlog.nl | www.biohorizonscamlog.nl

### Customer Service

Tel +31 (0)348 820 010 | info@camlog.nl

### Headquarters

CAMLOG Biotechnologies GmbH | Margarethenstr. 38 | 4053 Basel | Switzerland  
Phone +41 61 565 41 00 | Fax +41 61 565 41 01 | info@camlog.com | www.bhclgroup.com

Legal manufacturer Bone Scraper:  
OSUNG MND CO., LTD. | 57, Hwanggeum-ro 109 beon-gil | Yangchon-eup, Gimpo-si, Gyeonggi-do 10048 | Korea